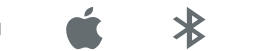
FitMaster AURA PRO

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FitMaster AURA PRO User Manual

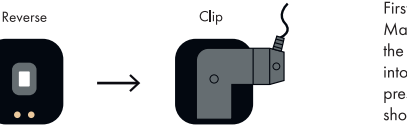
SMARTER RRA

Compatible System of Phone and BLE Version:

User Preparation

Please make sure the device is full charged before the first time using.

APP "LEFUN HEALTH" Download Scan the QR Code

Can download the app "LEFUN HEALTH" from the APPLE store OR Android market

Device Connection

First Long press the touch button to turn on the device. Make sure the smart phone bluetooth is on. Second, open the "LEFUN HEALTH" APP, press the top left corner, enter into the APP FUNCTION LIST, then press the APP icon, press "Search" choose the right device MAC number will show "Bluetooth pairing request" press "Pair".

Device Operation

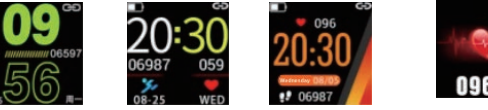
When the device sucessfully connected with the phone, the device will syn the phone time date. The APP will syn the device sports datas, heart rate monitoring datas, blood pressure datas and etc




Short press the device touch button to switch the function. Long press the touch button to trun ON/OFF.

Function Instruction

Home Display: Show Time, Date, Battery and etc;

Pedometer

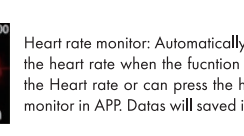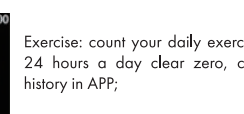
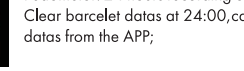
24 hours recording on device. Clear barcelet datas at 24:00,can check; datas from the APP;

Exercise: count your daily exercise Time, 24 hours a day clear zero, can View history in APP;

Heart rate monitor: Automatically measure the heart rate when the fuction switch to the Heart rate or can press the heart rate monitor in APP. Datas will saved in the APP.

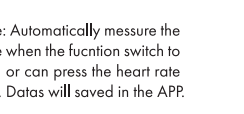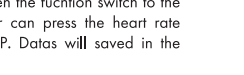SPO2 Test

Automatically measure the spO2 rate when the fuction switch to the Heart rate, or can press the heart rate monitor in APP. Datas will saved in the APP.

Blood pressure: Automatically measure the Blood pressure when the fuction switch to the Heart rate, or can press the heart rate monitor in APP. Datas will saved in the APP.

Smart Alarm

Find the "Smart Clock" on APP, set the time, press the time, can set the repeat mode, press complete to save the setting, the device will vibrate when time arrived.

Sleep monitor: Automatically record your sleep status with analyzing the deep sleep and light sleep hours, datas saved in app.

Sedentary Reminder

Open the function in APP and set the time duration, the device will vibrate.

Reminder for drinking: Open the function in APP and set the time duration, the device will vibrate.

Anti-lost: Open the function in APP, the device will vibrate when the phone is taken away.

Message Reminder: Open the function in APP, the device will vibrate when the phone comes a new message (SMS, QQ Facebook, Twitter,Whatsapp,Wechat).

Call Reminder: Open the function in APP, the device will vibrate when the phone comes.

Shake the device to take photo

Open the function in APP, then press the "Shake to take a photo", enter into the camera, shake the device to confirm. (Make sure the mobile Phone setting allow the APP to take photo and save the photo).

Find Device: Press the function in APP the device will vibrate.

Restore factory Settings: set the time to the previous day in the mobile APP or the data of the device can be cleared by synchronizing one day later;

*The company reserves the right to make changes to the contents of the specification without notice.